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EN - Safety Instructions for 3-Layer Protective Mask

The protective mask is designed to protect against dust, dirt, and viral infections. Use the mask only as intended – to protect the airways in risky environments such as public spaces, hospitals, or when in contact with sick individuals. Before putting on the mask, wash your hands with water and soap, or use an alcohol-based hand sanitizer. Place the mask on your face so that it covers the nose and mouth, and ensure it fits snugly. Attach the ear straps so that the mask sits comfortably against your face. If the

mask has a plastic reinforcement, ensure that it is correctly positioned to mold to the shape of the nose and face. The mask is for single use only. After use, dispose of it immediately. Do not reuse the mask. If the mask becomes wet, damaged, or contaminated during use, replace it immediately. When putting on or taking off the mask, avoid touching the inner side of the mask to prevent contamination. If the mask causes skin irritation or other health issues while wearing, stop using it and consult a doctor. If the mask comes into contact with contaminated fluids, replace it immediately. Store the mask in a dry, clean place, away from direct sunlight and heat sources. The mask has an antibacterial filtration efficiency of 99.5% and is designed to prevent the penetration of fluids and particles. However, it is not intended for protection against toxic gases or other chemicals. The mask is not intended for use by children under 2 years old. Children over this age should use the mask only under adult supervision. If you have doubts about the proper use or effectiveness of the mask, consult a professional or follow the instructions provided on the packaging.

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