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## **EN - ZINC-AIR BATTERIES (for hearing aids)**

### **Safety Instructions:**

Before use, insert the battery into the device as instructed by the hearing aid manufacturer. Store the batteries in a dry, cool place. Avoid extreme temperatures and humidity. The battery is for single use only. Do not recharge it, as it is not designed for recharging. Zinc-air batteries are not suitable for devices that require rechargeable batteries (e.g., Ni-MH batteries). Only use zinc-air batteries that are specifically intended for hearing aids. Do not attempt to disassemble, short-circuit, or subject the battery to impacts or pressure. Doing so may cause damage, which could lead to the release of chemicals or the generation of heat. Keep batteries out of reach of children. If swallowed, seek medical attention immediately. Do not use the battery if it is damaged, or if any electrolyte leakage occurs. Do not immerse batteries in water or seawater. The battery is intended for use in hearing aids only. When using, ensure correct polarity. If the battery emits an unusual odor, changes temperature, or leaks

electrolyte, immediately remove it from the device and dispose of it according to local regulations. Batteries should be stored in a dry, cool place, away from direct sunlight. Do not be alarmed if the hearing aid emits a sound when changing the battery. This is normal if the contact is correctly established. Always follow the hearing aid manufacturer's instructions when changing the battery.

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